


DNA Sample Collection Procedure

Please read through first, then follow instructions carefully.

Step 1: Label Swab Envelope

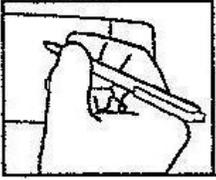

Remove one envelope of buccal swabs from the collection kit. Be sure to use the appropriate envelope for each party tested (ie. use the "Child" envelope for the child). If the "Additional Tested Party" envelope is used, indicate the relationship of the party being tested. (ie. grandfather, aunt, etc.).

Put the correct name on the swab envelope. Be sure to provide race information: it is necessary for statistical calculations.

Step 2: Collect Samples Using Swabs




Open the wrapper at the end opposite from the polyester tip. Remove one swab from the package, **taking care not to touch the polyester tip.**

Open mouth wide. Insert a swab into the mouth, and rub it firmly back and forth, up and down on the inside of the cheek **for a full 30 seconds.** Be sure to rotate the swab while rubbing - it is important to collect cells on the entire swab.

NOTE: insufficient DNA in the collected sample may require recollection and delay test results.

Step 3: Place Used Swabs in Envelope

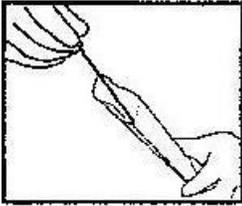

Remove the swab from the mouth. Hold the tip of the swab in the envelope. Press the opposite end with your thumb, and the sample end will be released into the envelope. Seal and label.

Step 4: Follow Steps 1-3 for Each Person

Collect samples from any other tested parties following Steps 1-3. Remember to label each DNA sample correctly and use one swabs provided for each person.

Step 5: Complete and Sign the Client Information Form

Step 6: Package Samples in Return Envelope

Place all the collected parties' swab envelopes, along with the completed Client Information Form, in the envelope provided.

Send the completed test to Orchid Pro-DNA 3885 Industriel Blvd Laval, Quebec H7L 4S3

Please Call [1-877-777-4004](tel:1-877-777-4004) to set up your file, and provide detailed information on all parties tested.

For any questions or if you need any documents please visit

Paternity Testing Centres of Canada

www.paternitycanada.com

info@paternitycanada.com